


TECHNIQUE CARD

ProUltra[®]  PIEZOFLOW[™]
ULTRASONIC IRRIGATION NEEDLE

DENSPLY
TULSA DENTAL
SPECIALTIES

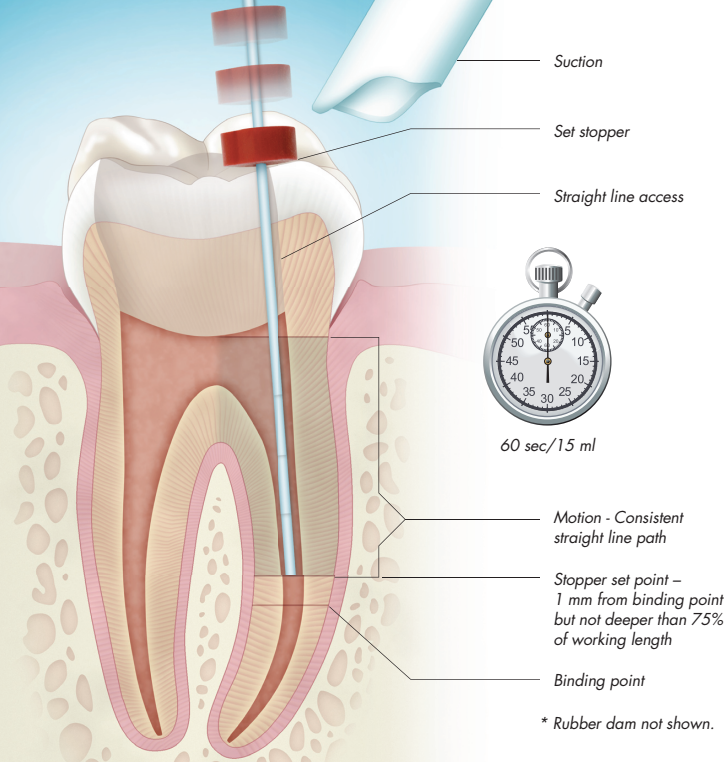

Utilize personal protective equipment and fully instrument and use a rubber dam before irrigation – Some additional orifice shaping may be required (see DFU).

- Alcohol wipe ProUltra® PiezoFlow™ Ultrasonic Irrigation Needle, wrench tighten onto ultrasonic handpiece
- Set the ProUltra® PiezoFlow™ Ultrasonic Irrigation Needle working length by trial fitting the needle into canal
 - **Set stopper 1 mm short of binding – no deeper than 75% of working length – needle should never be taken to apex**
- Attach minimum 20 ml syringe full of irrigant to PiezoFlow Luer Lock
- Set ProUltra® Piezo Booster to power level #5 – do not adjust
- **Place inactive ProUltra® PiezoFlow™ Ultrasonic Irrigation Needle into canal**
- **Begin irrigant delivery**, after irrigant has filled canal:
 - **Activate ultrasonic booster**
 - **Begin continuous suction** at access opening
 - **Move needle up and down** within canal – gently work in a straight line from top of canal to stopper setting
 - Ultrasonically irrigate for **1 minute with flow of 15 ml/min** per canal (NaOCL 6%)
- **Cease irrigant flow** before changing canals or removing needle

IMPORTANT INFORMATION

- Not for use on roots that have immature or open/perforated apices
- Not for use with chloroform
- Do not operate without flowing irrigant
- Cease flow of irrigant before removing needle from canal or changing canals
- Do not apply excessive force on the syringe
- Do not over-insert or bind tip into the canal
- Working times given in this technique card are for NaOCl (6%)
- Single patient use only

DENTSPLY
TULSA DENTAL
SPECIALTIES

Endodontics & Implants

*Our Quality. Your Success.
Their Satisfaction.*

DENTSPLY Tulsa Dental Specialties

DENTSPLY International, Inc.

608 Rolling Hills Dr.

Johnson City, TN 37604

1-800-662-1202

1-800-597-2779 (fax)

www.tulsadentalspecialties.com



Do Not Reuse



Non-Returnable If Opened



See Directions For Use

For Dental Use Only Rx only

Manufactured Under US Patent Numbers: xxxxxxxx, xxxxxxxx and other patents pending.

© 2009 DENTSPLY International, Inc. PUPFTC08/09